



## BENEFITS

- Can be used when infants are present
- Safe for surfaces, even the most fragile
- Bactericidal, yeasticidal et virucidal
- Foodstuff contact approved



Formule douce



## APPLICATION

Cleaning and disinfection of non-invasive medical devices (equipment, etc.) by contact.

Researched especially for neonatal services and, more specifically, for disinfecting sensitive surfaces (sterile bubbles, insulators, incubators, etc.) when infants are present.

IIA CLASS.

## INSTRUCTIONS FOR USE

Spray or apply by wiping BACTYNEA PRET A L'EMPLOI. Let dry. Do not rinse except in case of contact with surfaces that may come in contact with foodstuffs. Does not replace disinfection by soaking.

Complies with the decree of 19/12/13 amending the decree of 08/09/1999 concerning products for cleaning materials and articles intended to come into contact with foodstuffs.

## CHARACTERISTICS

Appearance:	liquid
Colour:	colourless
Perfume:	perfume free
Density:	= 1
pH:	low base
Expiration date:	2 years
Storage:	4°C - 35°C



## MICROBIOLOGICAL ACTIVITY

**Bactericidal** : EN 13727 (dirtiness conditions – 2 min – 20°C)

**Leviricidal** : EN 13624 (dirtiness conditions – 2 min – 20°C)

**Sporicidal** : EN 13704 (cleanliness conditions – 60 min – 20°C)

**Virucidal** : EN 14476 sur PRV, BVDV, H1N1, Herpès, Vaccinia virus, Rotavirus (dirtiness conditions – 2 min – 20°C), Norovirus (dirtiness conditions – 15 min – 20°C)

\* Swine Pseudorabies Virus (PRV) and Bovine Viral Diarrhea Virus (BVDV) are surrogate of HBV and HCV.

## PACKAGING

250ml bottle

Réf. LB920250

500ml bottle

Réf. LB920260

5L can

Réf. LB920270

## SECURITY

For professional use only.

For more information please refer to the MSDS.

