

BACTINYL LINGETTES DESINFECTANTES INODORE SINGLE USE ODOURLESS DISINFECTANT WIPES



BENEFITS

- Safe for surfaces even the most fragile
- Bactericidal, yeasticidal and virucidal
- Food contact agreement



Formule douce



APPLICATION

Single use wipes impregnated with disinfectant solution for cleaning and disinfection by contact of non-invasive medical devices, instrumentation, medical equipment and equipment.
Medical device. CLASS IIA.



INSTRUCTIONS FOR USE

Rub the surfaces for with BACTINYL LINGETTES DESINFECTANTES INODORES. Respect the action time, after the passage of the wipe, according to the efficiency desired. Let dry. Do not rinse except in case of contact with surfaces that may come into contact with foodstuffs.

Close the box after use for the impregnation rate is optimum to the last wipe. Do not replace disinfection by soaking.

Complies with the decree of 19/12/13 amending the decree of 08/09/1999 concerning products for cleaning materials and articles intended to come into contact with foodstuffs.

CHARACTERISTICS OF THE IMPREGNATING LIQUID

Appearance :	liquid
Colour :	colourless
Perfume :	odourless
Density :	= 1
pH :	low base
Expiration :	 2 years
Stockage :	 4°C - 35°C

MICROBIOLOGICAL ACTIVITY OF THE IMPREGNATING LIQUID

Bactericidal : EN 13727 (dirtiness conditions – 1 min – 20°C), EN 14561 (dirtiness conditions – 5 min – 20°C), EN 16615 (dirtiness conditions – 2 min – 20°C), EN 16615 (dirtytness conditions – 2 min +/- 15 sec – 20°C)

Yeasticidal : EN 13624 (dirtiness conditions – 1 min – 20°C), EN 14562 (dirtiness conditions – 15 min – 20°C), EN 16615 (dirtiness conditions – 2 min +/- 15 sec – 20°C)

Sporicidal : EN 13704 on Bacillus Subtilis and Clostridium difficile (cleanliness conditions – 60 min – 20°C)

Virucidal : EN 14476 on PRV*, BVDV*, H1N1, Herpès, Vaccinia virus, Rotavirus (dirtiness conditions – 5 min – 20°C), Norovirus (dirtiness conditions – 15 min – 20°C), HIV (dirtiness conditions – 60 sec – 20°C)

* Swine Pseudorabies Virus (PRV) and Bovine Viral Diarrhea Virus (BVDV) are surrogate of HBV and HCV.

PACKAGING

Tub of 120 wipes

Tub of 120 wipes (lareg size)

Bucket of 300 wipes

Réf. LB920630

Réf. LB920710B

Réf. LB920620

SECURITY

For professional use only.

For more information please refer to the MSDS.