

BACTINYL PRET A L'EMPLOI INODORE READY TO USE ODOURLESS DETERGENT DISINFECTANT



BENEFITS

- Safe for surfaces even the most fragile
- Bactericidal, yeasticidal and virucidal
- Food contact agreement



Formule douce



APPLICATION

Cleaning and disinfection by contact of non-invasive medical devices, instrumentation, medical equipment and equipment.

Medical device. CLASS IIA.

INSTRUCTIONS FOR USE

Applied by spraying or wiping BACTINYL PRET A L'EMPLOI INODORE. Respect the action time according to the efficiency desired. Let dry. Do not rinse except in case of contact with surfaces that may come into contact with foodstuffs.

Does not replace disinfection by soaking.

Complies with the decree of 19/12/13 amending the decree of 08/09/1999 concerning products for cleaning materials and articles intended to come into contact with foodstuffs.

CHARACTERISTICS

Appearance :	liquid
Colour :	colourless
Perfume :	odourless
Density :	= 1
pH :	low base
Expiration :	2 years
Stockage :	4°C - 35°C



MICROBIOLOGICAL ACTIVITY

Bactericidal : EN 13727 (dirtiness conditions – 5 min – 20°C), EN 14561 (dirtiness conditions – 5 min – 20°C)

Yeasticidal : EN 13624 (dirtiness conditions – 5 min – 20°C), EN 14562 (dirtiness conditions – 15 min – 20°C)

Sporicidal : EN 13704 (cleanliness conditions – 60 min – 20°C)

Virucidal : EN 14476 on PRV*, BVDV*, H1N1, Herpès, Vaccinia virus, Rotavirus (dirtiness conditions – 5 min – 20°C), Norovirus (dirtiness conditions – 15 min – 20°C)

* Swine Pseudorabies Virus (PRV) and Bovine Viral Diarrhea Virus (BVDV) are surrogate of HBV and HCV.

PACKAGING

250ml bottle

Réf. LB920810

500ml bottle

Réf. LB920820

5L can

Réf. LB920800

SECURITY

For professional use only.

For more information please refer to the MSDS.

